

### **Warranty**

This instrument is warranted for 90 days from the date of purchase for defects in materials and workmanship. Any defective instrument will be repaired or replaced at no charge, provided it is returned to the manufacturer sterile, prepaid, and marked with a valid Return Materials Authorization number (RMA). This warranty does not apply to misuse, abuse, or accidents during use. The manufacturer reserves the right to determine what constitutes defects in materials and workmanship. No other warranty is intended, expressed, or implied.

# Laparoscopic Peanut Dissector Handle Reusable

Item # SGH5/10\*

### **Instructions for Use**

#### **READ AND UNDERSTAND ALL INFORMATION CAREFULLY BEFORE USE**

In order to obtain optimum results, the following instructions should be followed. Otherwise, the instrument may not perform properly and/or injuries may result.

**Caution:** Federal law restricts this device to sale by or on the order of a physician.

Manufactured by:

**apollo**  
SURGICAL INDUSTRIES, INC.

741 Teaneck Rd | Teaneck NJ 07666  
Tel: (800) 749-4699 or (201) 739-6979  
Fax: (866) 731-0346  
[www.apollosurgicalinc.com](http://www.apollosurgicalinc.com)

*\*For Use With Laparoscopic Peanut Dissector Tip (Item #SG5/10)*

## INSTRUCTIONS FOR USE

### *Indications for Use*

This instrument is intended for use during surgical endoscopy with a Laparoscopic Peanut Dissector Tip (Item #SG5/10) for endo-dissection.

### *Description of Use*

This instrument is to be used by, or under the supervision of a surgeon performing endoscopic surgery. The Laparoscopic Peanut Dissector is intended for use in laparoscopic and thoracoscopic surgical procedures:

For blotting and/or sponging tissues.

For assisting in defining surgical planes and dissecting between them to separate tissues such as laparoscopic hernia repair, cholecystectomy, lysis of adhesions, and fundoplication.

For retracting organs atraumatically.

For gentle manipulation of tissues and/or organs.

For procedures that require suction, irrigation, blunt dissection and electrocoagulation in a variable sequence, such as laparoscopic cholecystectomy.

The Laparoscopic Peanut Dissector is intended to perform all the preceding functions without any instrument exchanges.

## PRECAUTIONS AND WARNINGS

### *General*

1. This instrument is packaged non-sterile and must be sterilized before use.
2. Inspect the product before use.
3. This product is intended for use with a Laparoscopic Peanut Dissector Tip (Item #SG5/10) disposable gauze dissector tip.

### *Cleaning Method*

1. Immediately after use, disassemble the instrument and discard the single use tip as hazardous waste. Then place in a tray or container and cover them with a moistened towel or clean distilled water.
2. Prepare the neutral-pH enzymatic detergent (safe for use with metal instruments) according to detergent manufacturer's recommendations. **Do not use Glutaraldehyde.**
3. Soak the entire device in neutral-pH enzymatic detergent for three(3) minutes
4. scrub the surrounding areas with a soft bristle brush and neutral-pH enzymatic detergent.
5. Rinse the entire device under running water for a minimum of one minute.
6. Inspect the device for blood and debris, repeat steps 1-5 as necessary.
7. Confirm the instrument is dry and wrap for sterilization.


### *Sterilization Method*

#### STEAM STERILIZATION

Type of Autoclave Sterilization Method	Temperature of Gas Content	Minimum Exposure Time	Condition of Instrument
Gravity Displacement	270-272°F (132-133°C)	20 minutes	Wrapped
Pre-vacuumed	270-272°F (132-133°C)	4 minutes (10 minute dry time)	Wrapped

\*Do not exceed 285°F/140°C

CE  
0120

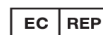

Symbol Medical GmbH  
Schwarzschildstraße 6, D-12489 Berlin, Germany  
Email: mmeq\_eu@modernmedical.com.hk



Exclusively manufactured by Modern Medical Equipment Mfg. Ltd.  
5fl, 35 Tai Lin Pai Rd., Kwai Chung, Hong Kong

## **Warranty**

This instrument is warranted for 90 days from the date of purchase for defects in materials and workmanship. Any defective instrument will be replaced at no charge, provided it is returned to the manufacturer sterile, prepaid, and marked with a valid Return Materials Authorization number (RMA). This warranty does not apply to misuse, abuse, or accidents during use. The manufacturer reserves the right to determine what constitutes defects in materials and workmanship. No other warranty is intended, expressed, or implied.

# Laparoscopic Peanut Dissector Tip

## Disposable

Item # SG5/10\*

## **Instructions for Use**

### **READ AND UNDERSTAND ALL INFORMATION CAREFULLY BEFORE USE**

In order to obtain optimum results, the following instructions should be followed. Otherwise, the instrument may not perform properly and/or injuries may result.

**Caution:** Federal law restricts this device to sale by or on the order of a physician.

Manufactured by:

**apollo**  
SURGICAL INDUSTRIES, INC.

741 Teaneck Rd | Teaneck NJ 07666  
Tel: (800) 749-4699 or (201) 739-6979  
Fax: (866)731-0346  
[www.apollosurgicalinc.com](http://www.apollosurgicalinc.com)

*\*For Use With Laparoscopic Peanut Dissector Reusable Handle  
(Item #SGH5/10)*

## INSTRUCTIONS FOR USE

### *Indications for Use*

This instrument is intended for use during surgical endoscopy, threaded firmly to the Laparoscopic Peanut Dissector Reusable Handle (Item #SGH5/10) for endo-dissection.

### *Description of Use*

This instrument is to be used by, or under the supervision of a surgeon performing endoscopic surgery. The Laparoscopic Peanut Dissector is intended for use in laparoscopic and thoracoscopic surgical procedures:

For blotting and/or sponging tissues.

For assisting in defining surgical planes and dissecting between them to separate tissues such as laparoscopic hernia repair, cholecystectomy, lysis of adhesions, and fundoplication.

For retracting organs atraumatically.

For gentle manipulation of tissues and/or organs.

For performing procedures that require suction, irrigation, blunt dissection and electrocoagulation in a variable sequence, such as laparoscopic cholecystectomy.

The Laparoscopic Peanut Dissector is intended to perform all the preceding functions without any instrument exchanges.

## PRECAUTIONS AND WARNINGS

### *General*

1. This instrument is packaged sterile and ready for use.
2. Inspect the product before use.
3. This product is intended for use with the Laparoscopic Peanut Dissector Reusable Handle (Item # SGH5/10).

### *Disposal*

After use, the Laparoscopic Peanut Dissector disposable tip must be discarded as medical waste in accordance with the applicable law.

Manufactured for:  
Apollo Surgical Industries, Inc.  
741 Teaneck Rd | Teaneck NJ 07666  
Tel: (800) 749-4699 or (201) 739-6979  
Fax (866) 731-0346  
[www.apollosurgicalinc.com](http://www.apollosurgicalinc.com)

CE  
0120



Symbol Medical GmbH  
Schwarzschildstraße 6, D-12489 Berlin, Germany  
Email: [mmeq\\_eu@modernmedical.com.hk](mailto:mmeq_eu@modernmedical.com.hk)



Exclusively manufactured by Modern Medical Equipment Mfg. Ltd.  
5fl, 35 Tai Lin Pai Rd., Kwai Chung, Hong Kong

## **Warranty**

This instrument is warranted for 90 days from the date of purchase for defects in materials and workmanship. Any defective instrument will be replaced at no charge, provided it is returned to the manufacturer sterile, prepaid, and marked with a valid Return Materials Authorization number (RMA). This warranty does not apply to misuse, abuse, or accidents during use. The manufacturer reserves the right to determine what constitutes defects in materials and workmanship. No other warranty is intended, expressed, or implied.

# Laparoscopic Peanut Dissector

## Disposable

Item # TDSG5/10

## **Instructions for Use**

**READ AND UNDERSTAND ALL INFORMATION  
CAREFULLY BEFORE USE**

In order to obtain optimum results, the following instructions should be followed. Otherwise, the instrument may not perform properly and/or injuries may result.

**Caution:** Federal law restricts this device to sale by or on the order of a physician.

Manufactured by:

**apollo**  
SURGICAL INDUSTRIES, INC.

741 Teaneck Rd | Teaneck NJ 07666  
Tel: (800) 749-4699 or (201) 739-6979  
Fax: (866) 731-0346  
[www.apollosurgicalinc.com](http://www.apollosurgicalinc.com)

## INSTRUCTIONS FOR USE

### *Indications for Use*

This instrument is intended for use during surgical endoscopy for endo-dissection, retraction, and gentle manipulation.

### *Description of Use*

This instrument is to be used by, or under the supervision of a surgeon performing endoscopic surgery. The disposable Laparoscopic Peanut Dissector is intended for use in laparoscopic and thoracoscopic surgical procedures:

For blotting and/or sponging tissues.

For assisting in defining surgical planes and dissecting between them to separate tissues such as laparoscopic hernia repair, cholecystectomy, lysis of adhesions, and fundoplication.

For retracting organs atraumatically.

For procedures that require suction, irrigation, blunt dissection and electrocoagulation in a variable sequence, such as laparoscopic cholecystectomy.

For atraumatic manipulation of tissues and/or organs.

The disposable Laparoscopic Peanut Dissector is intended to perform all the preceding functions without any instrument exchanges.

## PRECAUTIONS AND WARNINGS

### *General*

1. This instrument is packaged sterile and ready for use.
2. Inspect the product before use.
3. This instrument is totally disposable and must be discarded after a single use.

### *Disposal*

After use, the disposable Laparoscopic Peanut Dissector must be discarded as medical waste in accordance with the applicable law.

Manufactured for:  
Apollo Surgical Industries, Inc.  
741 Teaneck Rd | Teaneck NJ 07666  
Tel: (800) 749-4699 or (201) 739-6979  
Fax (866) 731-0346  
[www.apollosurgicalinc.com](http://www.apollosurgicalinc.com)

CE  
0120



Symbol Medical GmbH  
Schwarzschildstraße 6, D-12489 Berlin, Germany  
Email: [mmeq\\_eu@modernmedical.com.hk](mailto:mmeq_eu@modernmedical.com.hk)



Exclusively manufactured by Modern Medical Equipment Mfg. Ltd.  
5fl, 35 Tai Lin Pai Rd., Kwai Chung, Hong Kong