

Warranty

This instrument is warranted for 90 days from the date of purchase for defects in materials and workmanship. Any defective instrument will be replaced at no charge, provided it is returned to the manufacturer sterile, prepaid, and marked with a valid Return Materials Authorization number (RMA). This warranty does not apply to misuse, abuse, or accidents during use. The manufacturer reserves the right to determine what constitutes defects in materials and workmanship. No other warranty is intended, expressed, or implied.

Laparoscopic "Peanut" Dissector

Disposable

Item # TDSG5/10

Instructions for Use

READ AND UNDERSTAND ALL INFORMATION CAREFULLY BEFORE USE

In order to obtain optimum results, the following instructions should be followed. Otherwise, the instrument may not perform properly and/or injuries may result.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Manufactured by:



741 Teaneck Rd, Teaneck NJ 07666

Phone: 201-833-5196

Fax: 866-731-0346

www.apollosurgicalinc.com

INSTRUCTIONS FOR USE

Indications for Use

This instrument is intended for use during surgical endoscopy for endo-dissection, retraction, and gentle manipulation.

Description of Use

This instrument is to be used by, or under the supervision of a surgeon performing endoscopic surgery. The disposable laparoscopic "Peanut" Dissector is intended for use in laparoscopic and thoracoscopic surgical procedures:

For blotting and/or sponging tissues.

For assisting in defining surgical planes and dissecting between them to separate tissues such as laparoscopic hernia repair, cholecystectomy, lysis of adhesions, and fundoplication.

For retracting organs atraumatically.

For procedures that require suction, irrigation, blunt dissection and electrocoagulation in a variable sequence, such as laparoscopic cholecystectomy.

For atraumatic manipulation of tissues and/or organs.

The disposable laparoscopic "Peanut" Dissector is intended to perform all the preceding functions without any instrument exchanges.

PRECAUTIONS AND WARNINGS

General

1. This instrument is packaged sterile and ready for use.

2. Inspect the product before use.

3. This instrument is totally disposable and must be discarded after a single use.

Disposal

After use, the disposable laparoscopic "Peanut" Dissector must be discarded as medical waste in accordance with the applicable law.

Warranty

This instrument is warranted for 90 days from the date of purchase for defects in materials and workmanship. Any defective instrument will be repaired or replaced at no charge, provided it is returned to the manufacturer sterile, prepaid, and marked with a valid Return Materials Authorization number (RMA). This warranty does not apply to misuse, abuse, or accidents during use. The manufacturer reserves the right to determine what constitutes defects in materials and workmanship. No other warranty is intended, expressed, or implied.

Laparoscopic "Peanut" Dissector Handle Reusable

Item # SGH5/10*

Instructions for Use

READ AND UNDERSTAND ALL INFORMATION CAREFULLY BEFORE USE

In order to obtain optimum results, the following instructions should be followed. Otherwise, the instrument may not perform properly and/or injuries may result.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Manufactured by:



741 Teaneck Rd, Teaneck NJ 07666

Phone: 201-833-5196

Fax: 866-731-0346

www.apollosurgicalinc.com

**For Use With "Peanut" Dissector Tip (Item #SG5/10)*

INSTRUCTIONS FOR USE

Indications for Use

This instrument is intended for use during surgical endoscopy with a laparoscopic "Peanut" Dissector Tip (Item #SG5/10) for endo-dissection.

Description of Use

This instrument is to be used by, or under the supervision of a surgeon performing endoscopic surgery. The laparoscopic "Peanut" Dissector is intended for use in laparoscopic and thoracoscopic surgical procedures:

For blotting and/or sponging tissues.

For assisting in defining surgical planes and dissecting between them to separate tissues such as laparoscopic hernia repair, cholecystectomy, lysis of adhesions, and fundoplication.

For retracting organs atraumatically.

For gentle manipulation of tissues and/or organs.

For procedures that require suction, irrigation, blunt dissection and electrocoagulation in a variable sequence, such as laparoscopic cholecystectomy.

The laparoscopic "Peanut" Dissector is intended to perform all the preceding functions without any instrument exchanges.

PRECAUTIONS AND WARNINGS

General

1. This instrument is packaged non-sterile and must be sterilized before use.

2. Inspect the product before use.

3. This product is intended for use with a laparoscopic "Peanut" Dissector Tip (Item #SG5/10) disposable gauze dissector tip.

Cleaning Method

1. Immediately after use, disassemble the instrument and discard the single use tip as hazardous waste. Then place in a tray or container and cover them with a moistened towel or clean distilled water.

2. To begin the decontamination process, wipe all visible debris from instrument and immerse in an approved enzymatic solution per the manufacturer's recommendation for time and dilution.

3. Rinse with cold tap water after the appropriate time and utilizing a small soft brush, clean the outside and inside lumen of the cannula. Be mindful of keeping the instrument immersed at all times in your cleaning solution.

4. Rinse with warm tap water to remove all detergent and contaminated residue.

5. If necessary, place the instrument in an ultrasonic bath containing sonic detergent. Refer to your manufacturer's instructions for proper use and duration of your ultrasonic bath.

6. Rinse, wipe dry, and visually inspect instrument for cleanliness.

7. Confirm the instrument is dry and wrap for sterilization.

Sterilization Method

This instrument may be steam autoclaved or sterilized with ethylene oxide in accordance with the sterilizer manufacturer's recommendations and, in accordance with hospital protocol for stainless steel hand instruments.

Warranty

This instrument is warranted for 90 days from the date of purchase for defects in materials and workmanship. Any defective instrument will be replaced at no charge, provided it is returned to the manufacturer sterile, prepaid, and marked with a valid Return Materials Authorization number (RMA). This warranty does not apply to misuse, abuse, or accidents during use. The manufacturer reserves the right to determine what constitutes defects in materials and workmanship. No other warranty is intended, expressed, or implied.

Laparoscopic "Peanut" Dissector Tip

Disposable

Item # SG5/10*

Instructions for Use

READ AND UNDERSTAND ALL INFORMATION CAREFULLY BEFORE USE

In order to obtain optimum results, the following instructions should be followed. Otherwise, the instrument may not perform properly and/or injuries may result.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Manufactured by:



741 Teaneck Rd, Teaneck NJ 07666

Phone: 201-833-5196

Fax: 866-731-0346

www.apollosurgicalinc.com

**For Use With "Peanut" Dissector Reusable Handle (Item #SGH5/10)*

INSTRUCTIONS FOR USE

Indications for Use

This instrument is intended for use during surgical endoscopy, threaded firmly to the Laparoscopic "Peanut" Reusable Handle (Item #SGH5/10) for endo-dissection.

Description of Use

This instrument is to be used by, or under the supervision of a surgeon performing endoscopic surgery. The Surgical Gentle Touch™ is intended for use in laparoscopic and thoracoscopic surgical procedures:

For blotting and/or sponging tissues.

For assisting in defining surgical planes and dissecting between them to separate tissues such as laparoscopic hernia repair, cholecystectomy, lysis of adhesions, and fundoplication.

For retracting organs atraumatically.

For performing procedures that require suction, irrigation, blunt dissection and electrocoagulation in a variable sequence, such as laparoscopic cholecystectomy. The Surgical Gentle Touch™ is intended to perform all the preceding functions without any instrument exchanges.

For atraumatic manipulation of tissues and/or organs.

3. This product is intended for use with the Laproscopic "Peanut" Reusable Handle (Item # SGH5/10).

Disposal

After use, the Laparoscopic "Peanut" Dissector disposable tip must be discarded as medical waste in accordance with the applicable law.

PRECAUTIONS AND WARNINGS

General

1. This instrument is packaged sterile and ready for use.
2. Inspect the product before use.